




QUICK START UP GUIDE

APOLLO

START UP INSTRUCTIONS


- ▶ Plug APOLLO into the mains using his charger. The mains port is situated near his left hip.

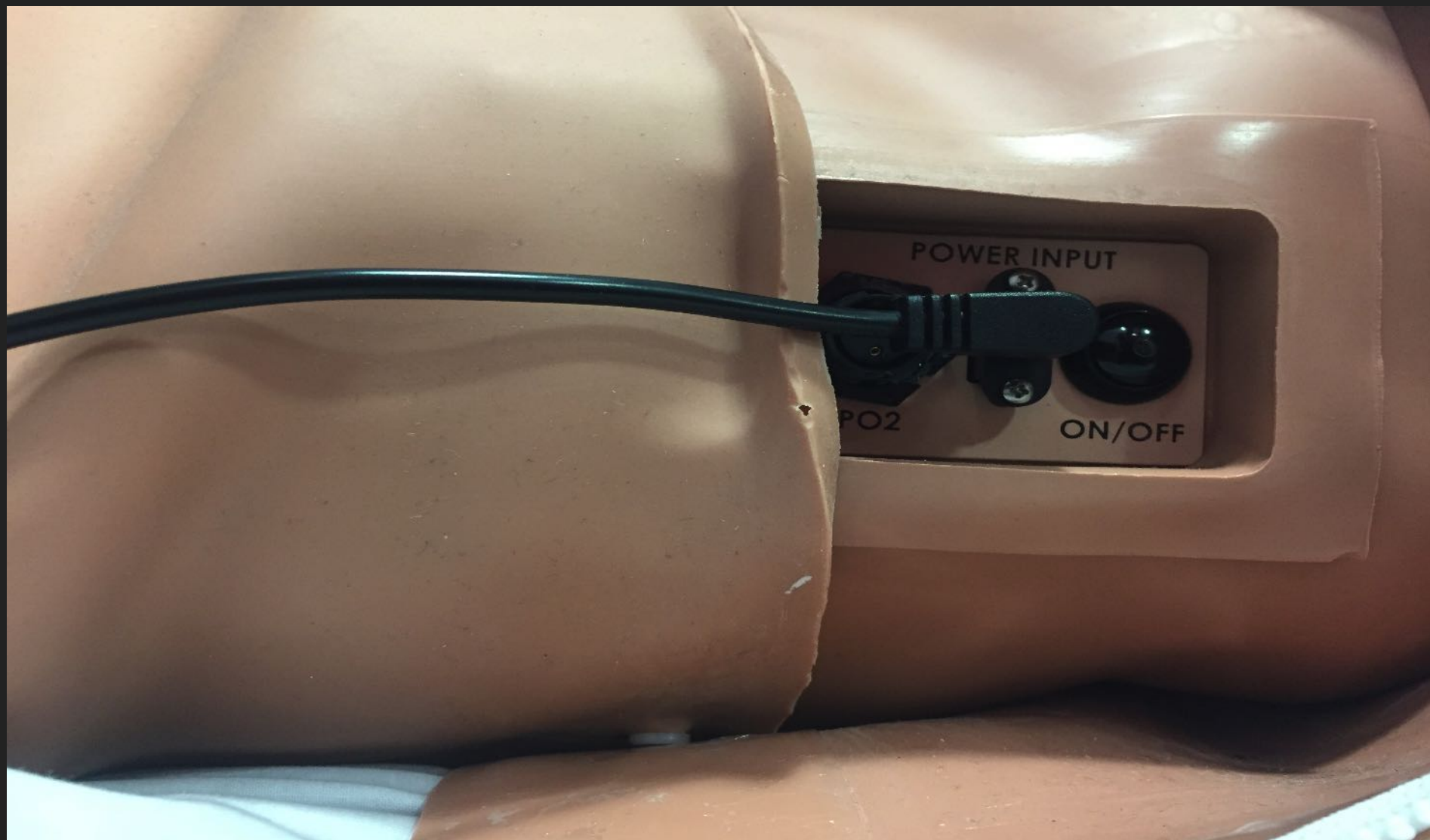


-
- ▶ Turn on APOLLO by pressing the black button next to where the power input port is located.
 - ▶ The button will flash green. Wait until you see the light turn to a constant green.





- ▶ Turn on the controller tablet by pressing the small button located on the top left hand edge.
- ▶ Open and turn on the MacBook Pro.
- ▶ When asked on the MacBook Pro Enter the password METIADMIN in capitals.
- ▶ Ensure both devices are connected to the **APP0261** network.

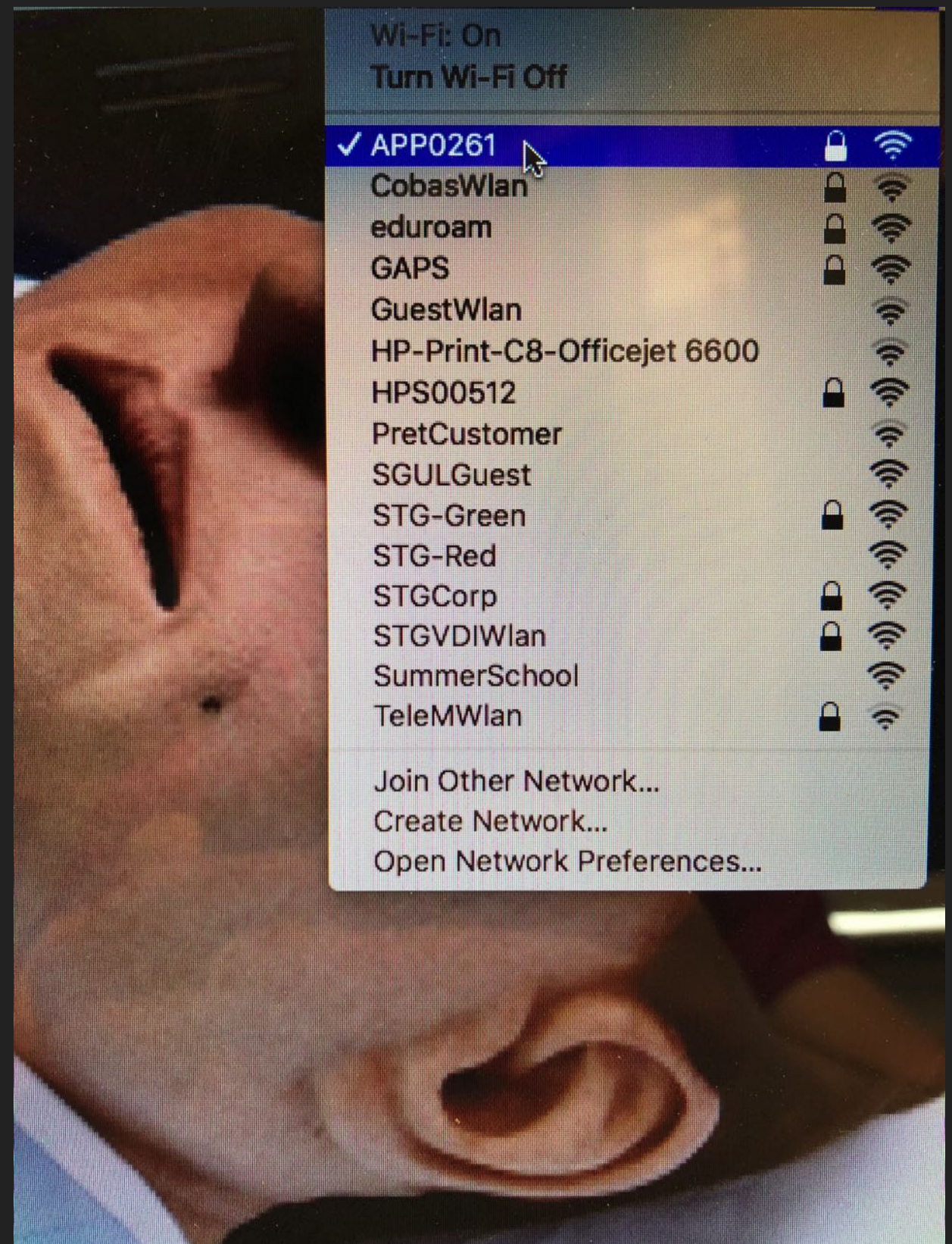



- ▶ On the controller tablet swipe your finger up the screen and then select the VIVO icon at the top right of the screen.
- ▶ Once VIVO has loaded you can select Run Simulated Clinical Experience (SCE)

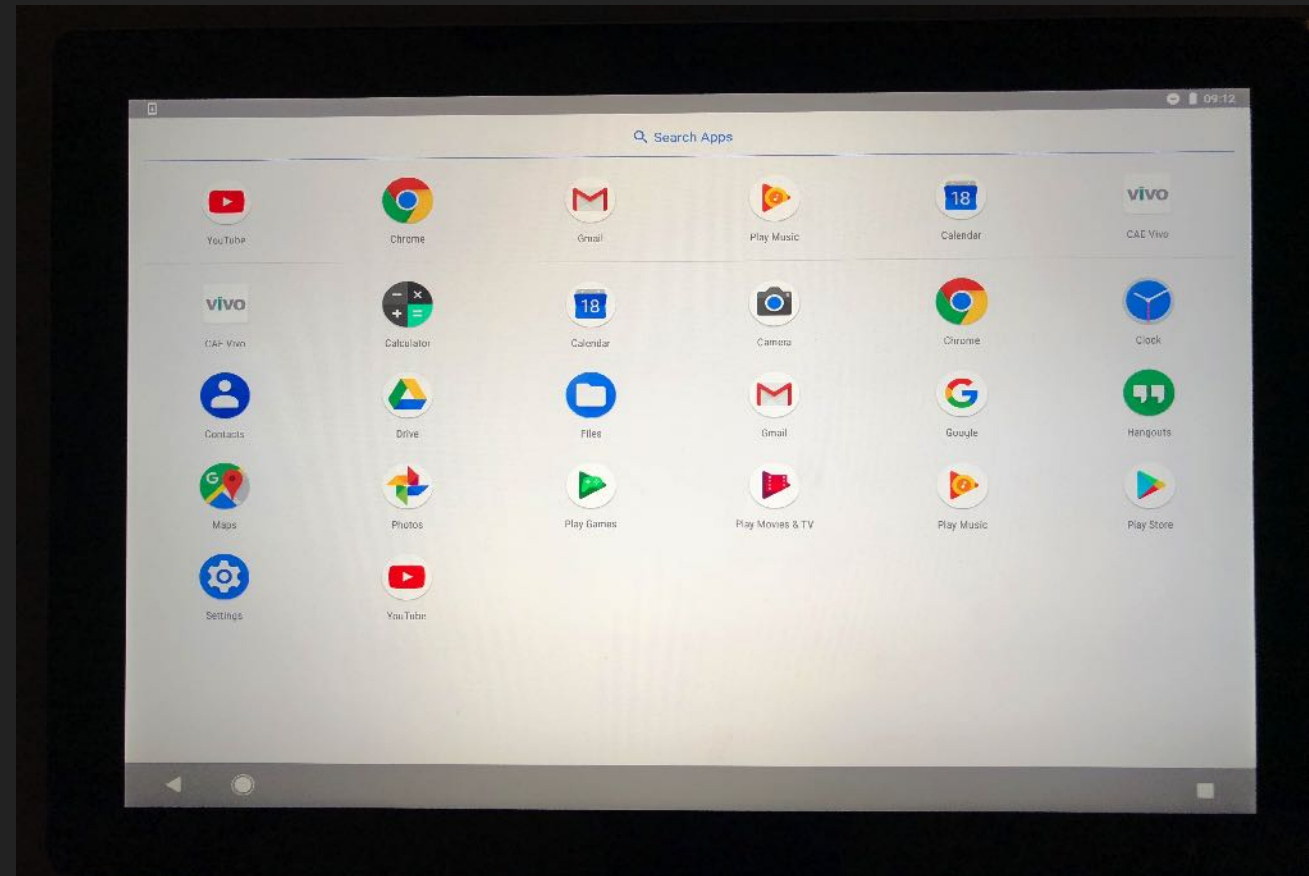


- ▶ You can then begin the simulation and edit the patient settings to fit your scenario.

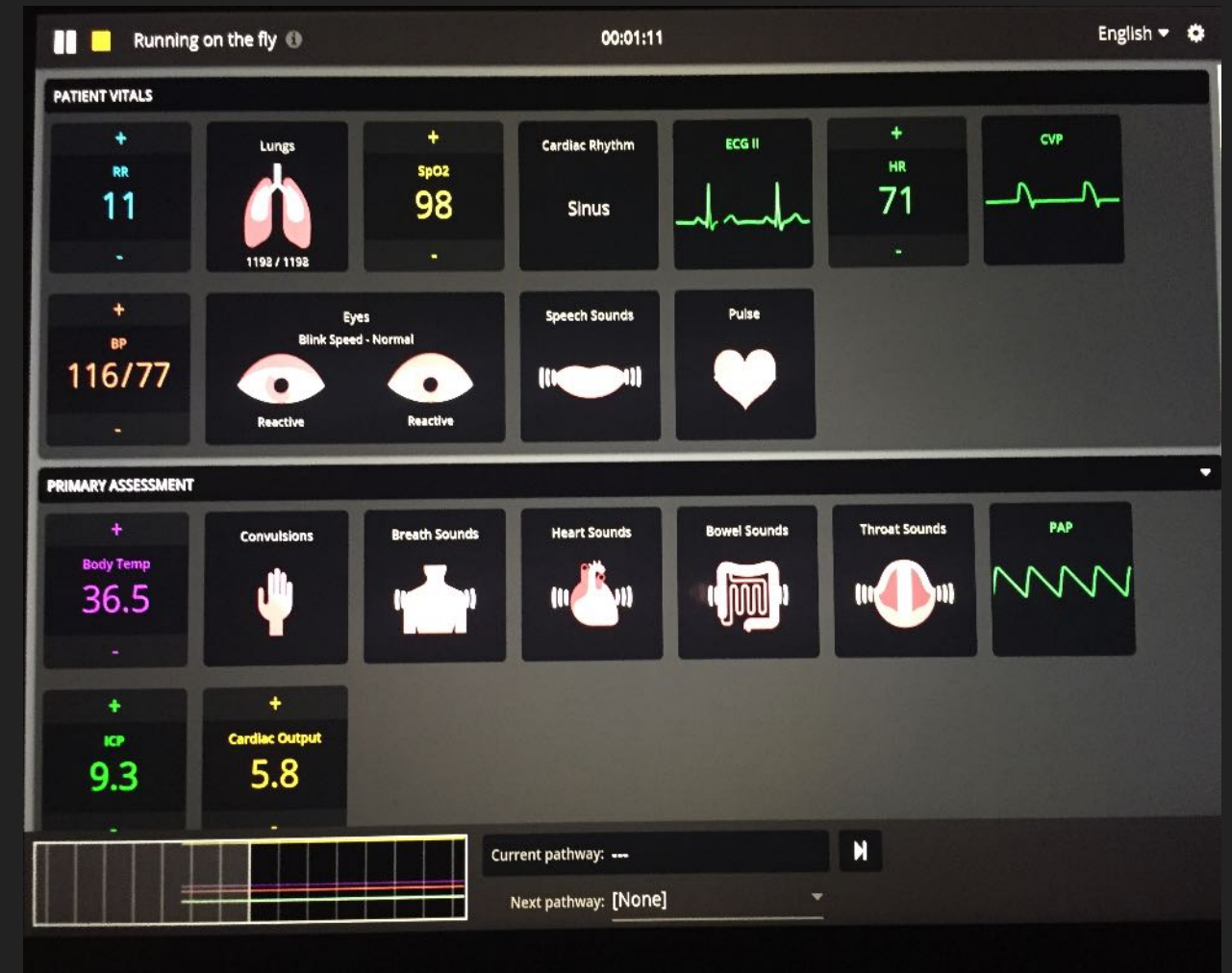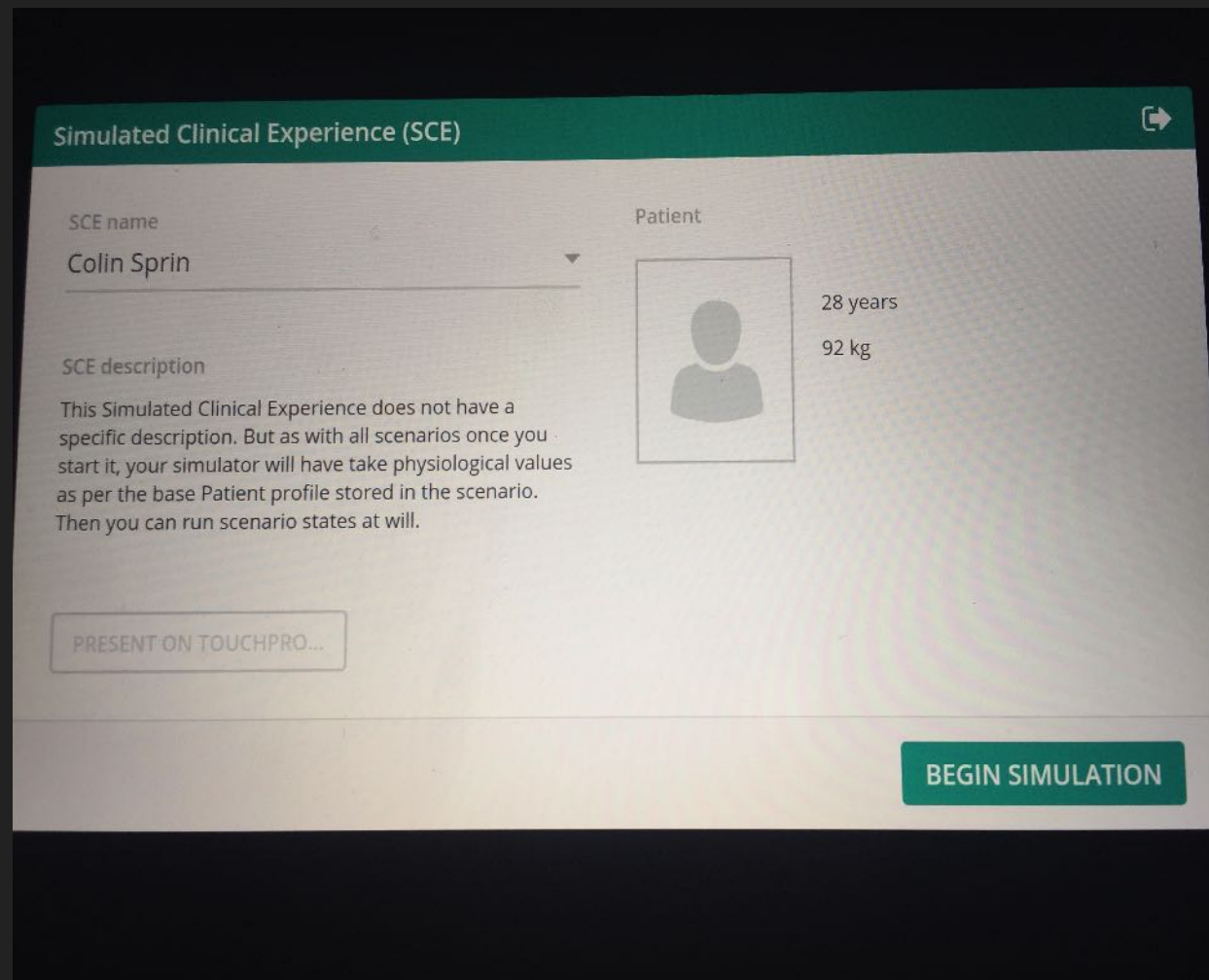

- ▶ On the MacBook Pro select safari located at the bottom of the screen.
- ▶ Then select the TouchPro APOLLO page.

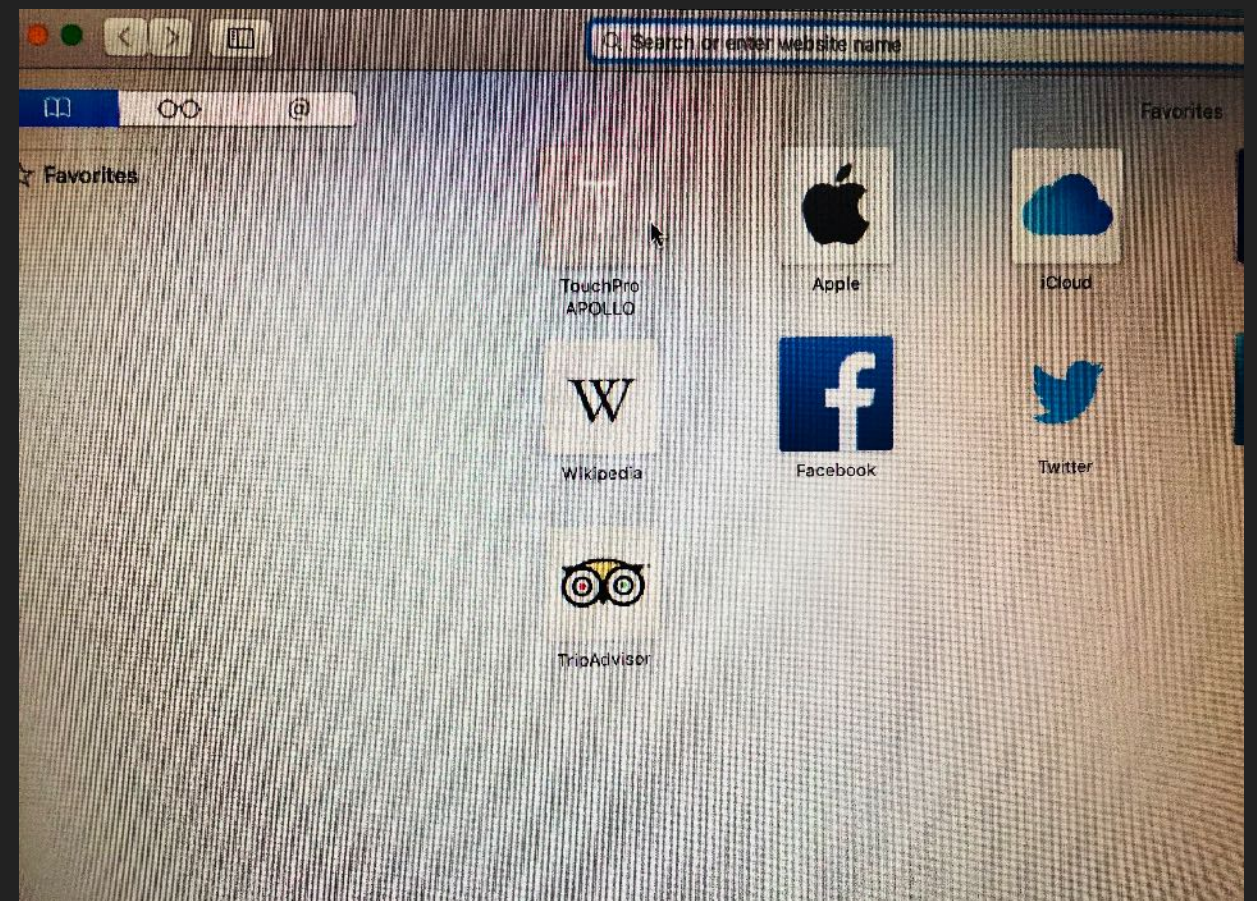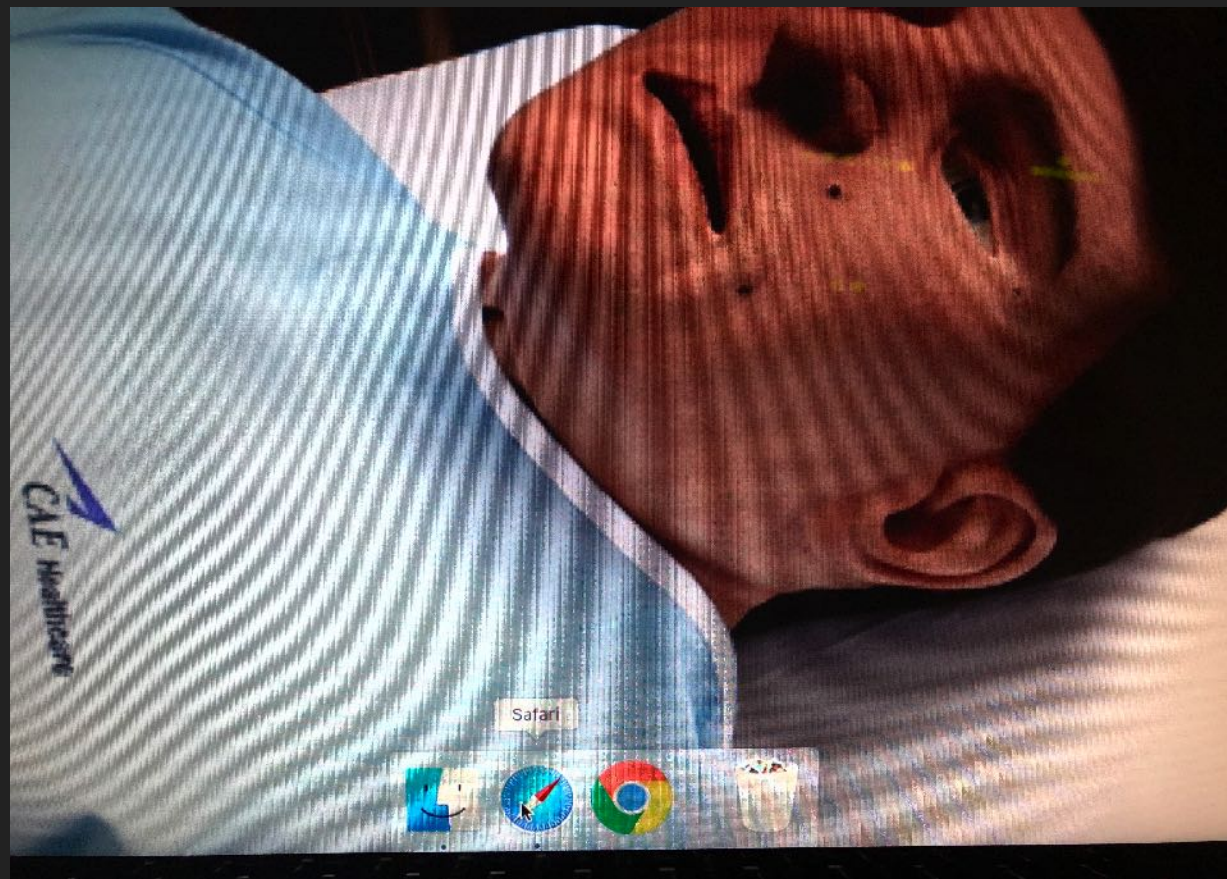


-
- ▶ Once the MUSE Welcome Page has opened you should select TouchPro Patient Monitor icon.

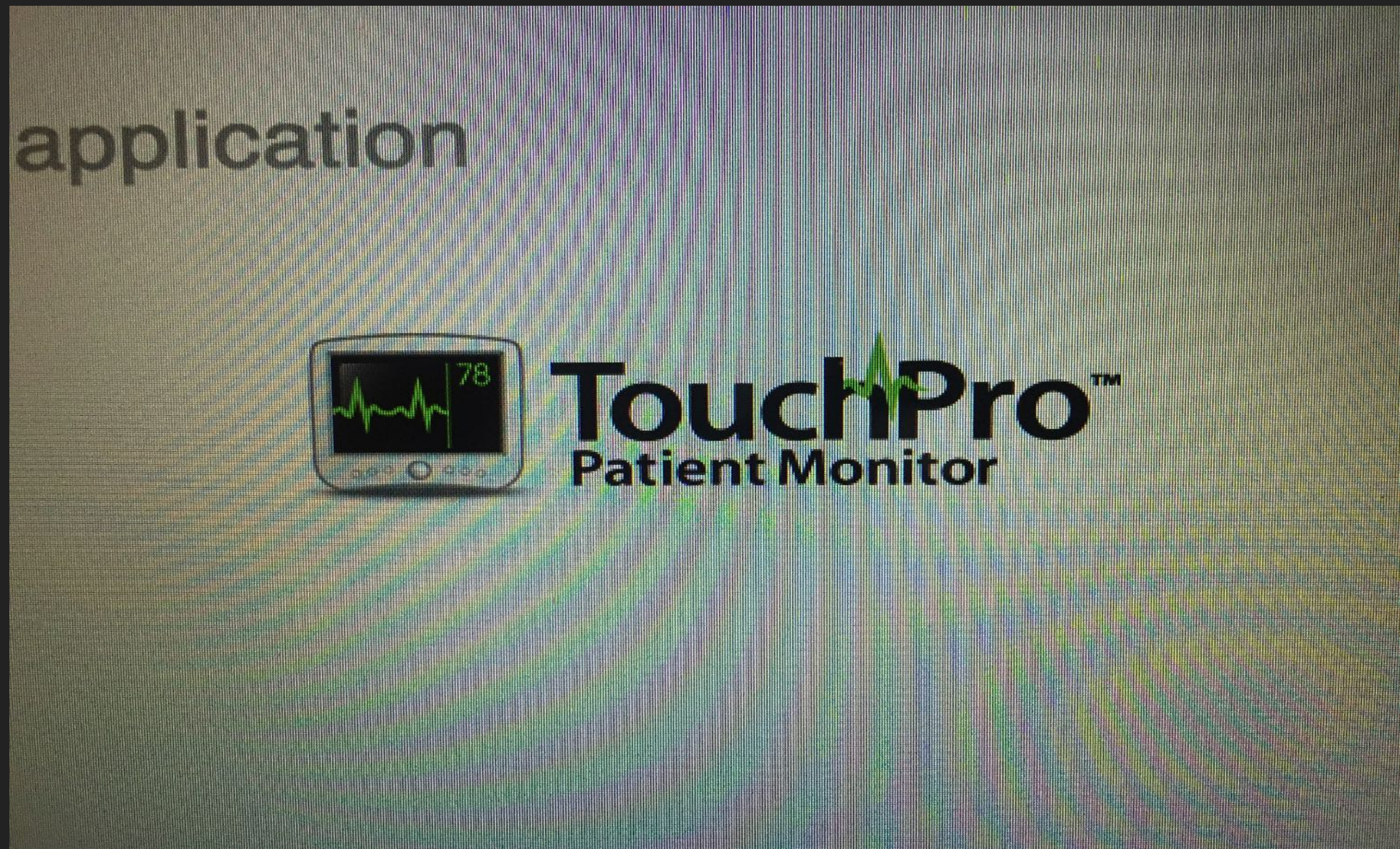


SHUT DOWN INSTRUCTIONS



- ▶ On the MacBook Pro click the settings button located at the bottom right of the screen and select shut down.

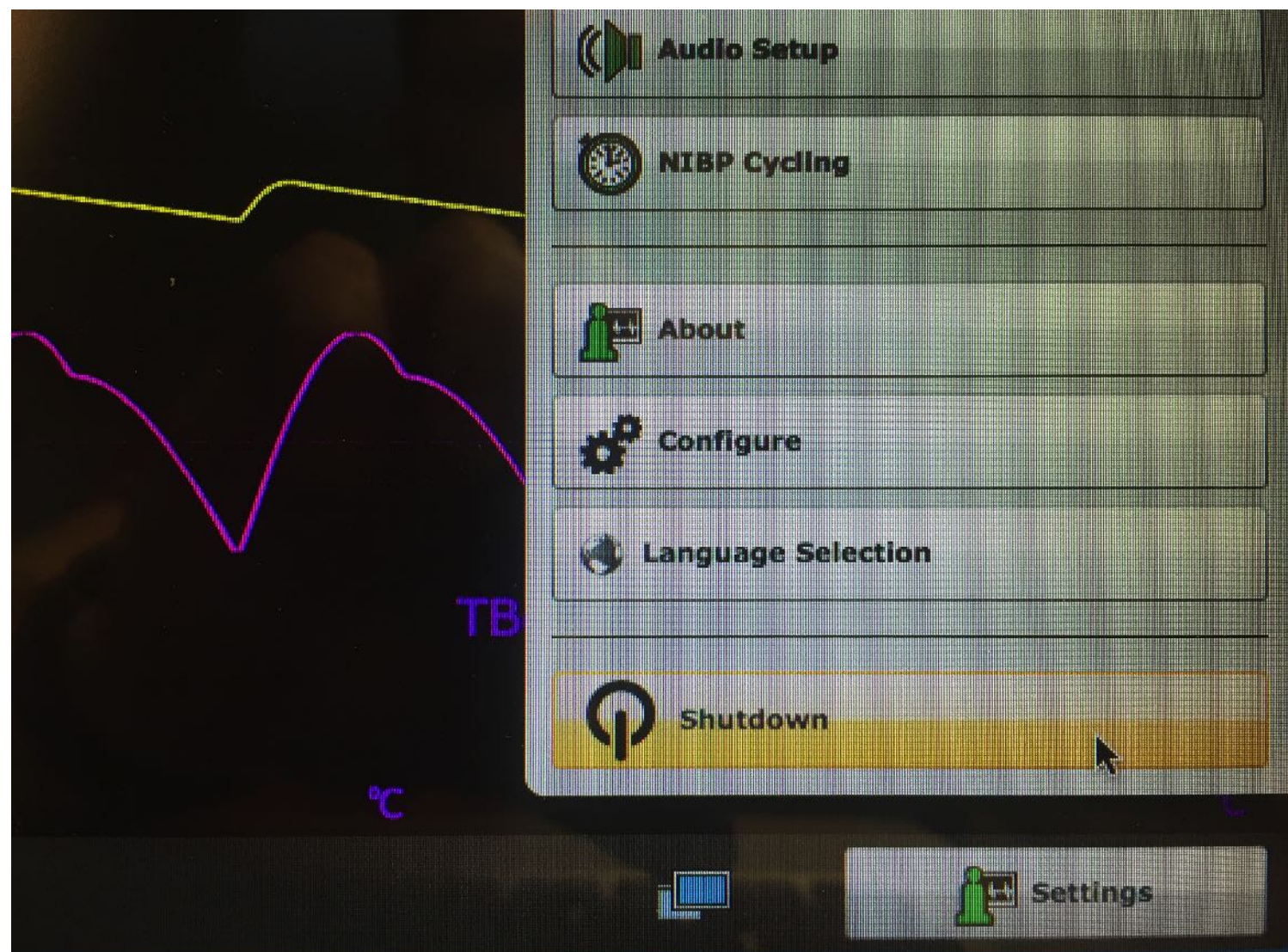


-
- ▶ Close MUSE and select the apple icon located at the top left of the screen. Select shut down from the drop down box.

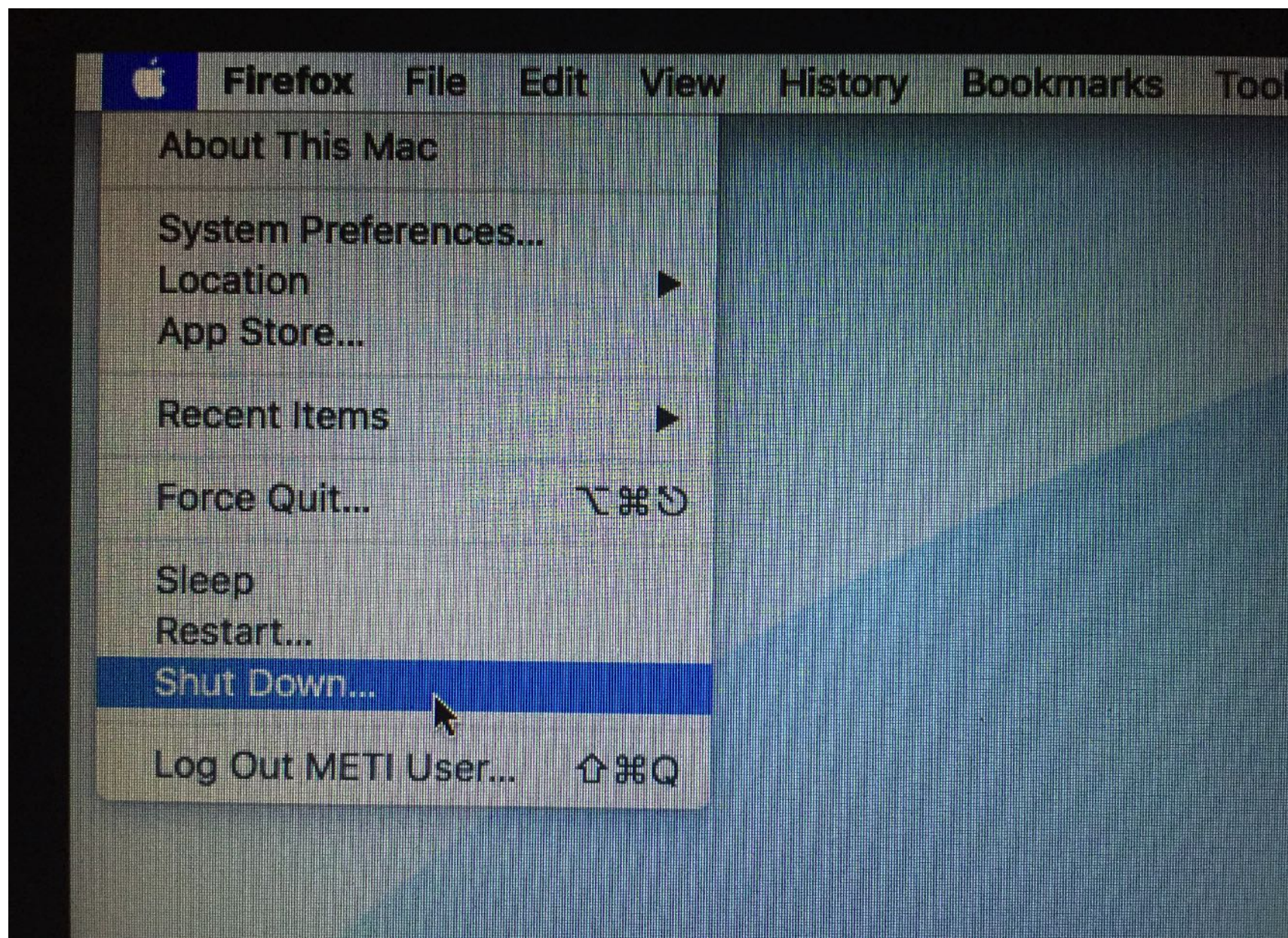


- ▶ To end the scenario on the controller tablet you must select the yellow square at the top left of the screen.
- ▶ Then drag your finger down the screen and select the square at the bottom right of the screen.

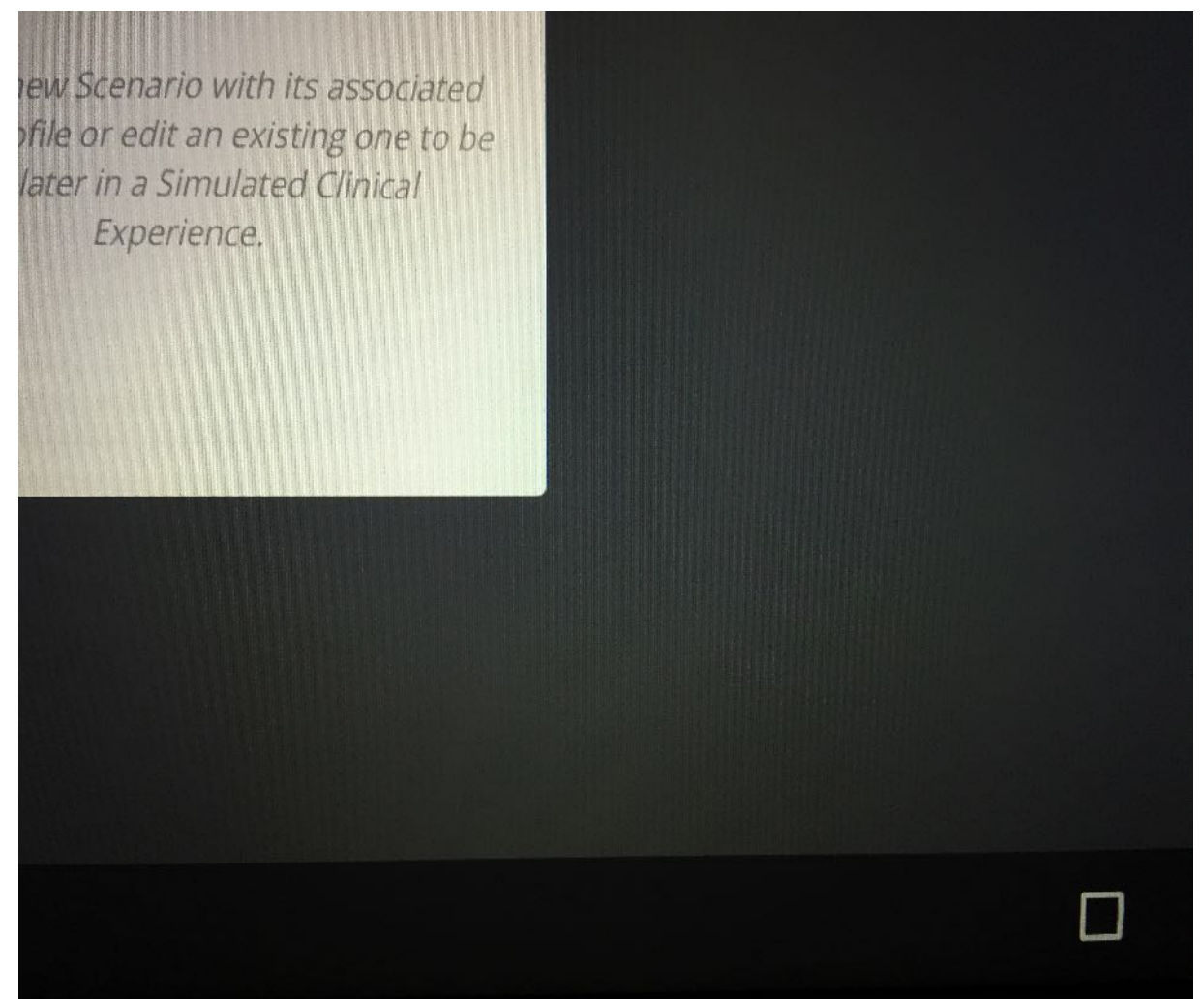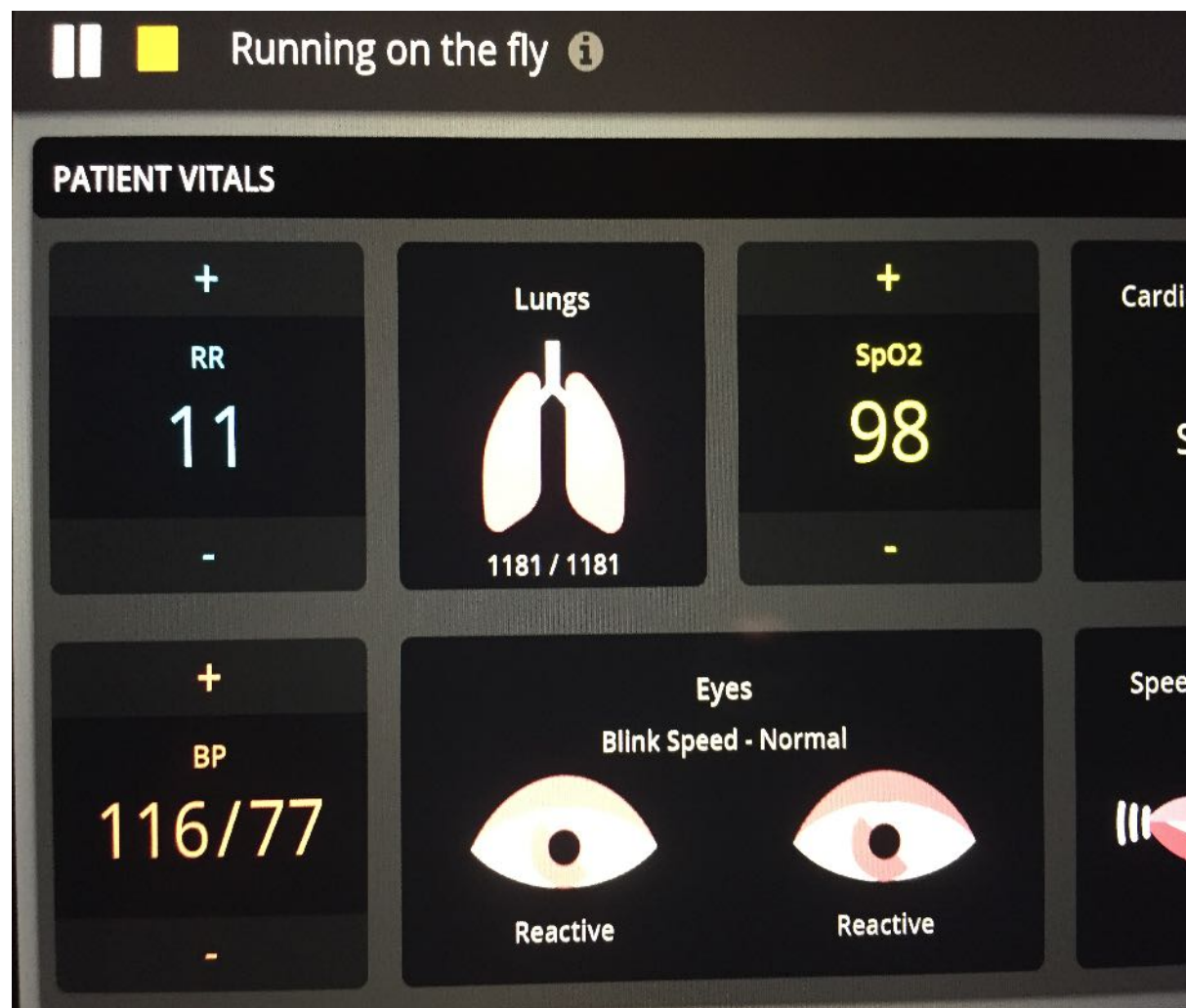

-
- ▶ Press and hold the power button on the controller tablet and then select power off.

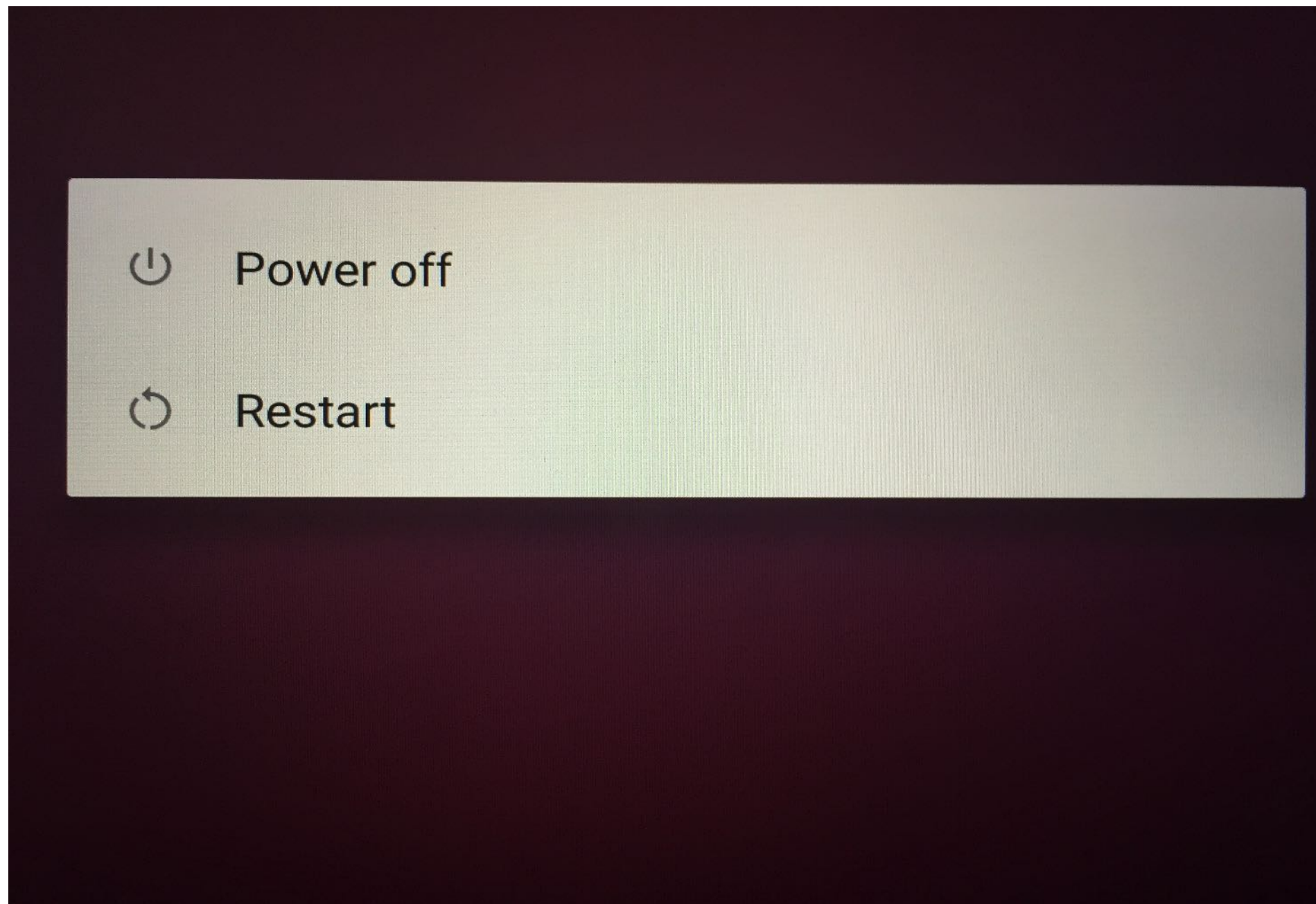

-
- ▶ Carefully unzip the abdominal skin on the left of APOLLO and press the power button for two seconds.
 - ▶ The button will blink. Allow around 30 seconds for APOLLO to shut down completely. Carefully return the skin in the correct position.

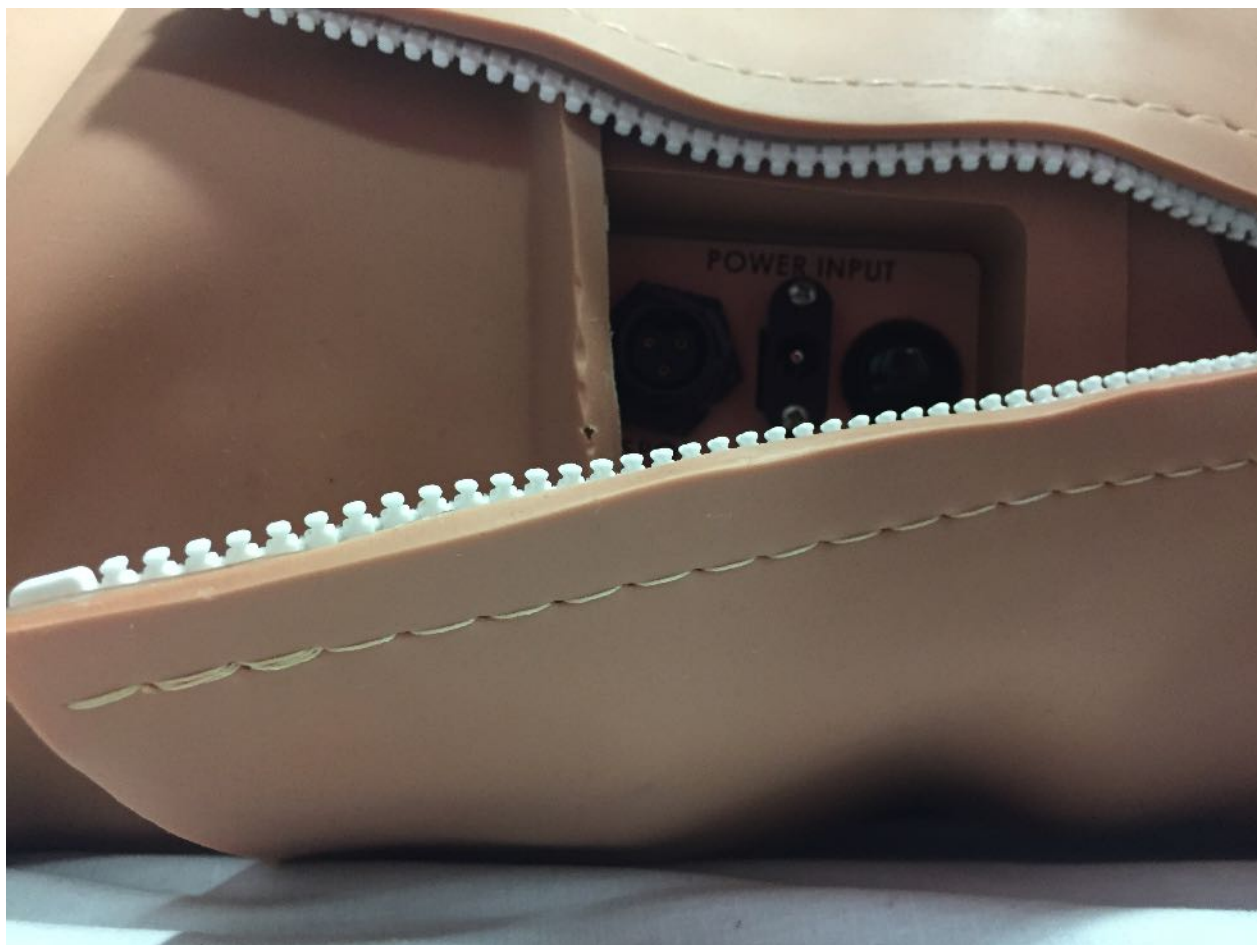

-
- ▶ Pack the laptops with all chargers into the bag provided.
 - ▶ Ensure that you also return any other loaned equipment back to GAPS.



**IF YOU HAVE ANY TECHNICAL ISSUES PLEASE
CONTACT US ON 1119.THANK YOU.**

