

REMEMBER: machine on, mask on: machine off, mask off

Dreamstation CPAP (all white)


No internal battery – mask use only



A

Single limb circuit

**Pre-made by
respiratory team –
can use tubing as
per Trilogy/ Vivo**

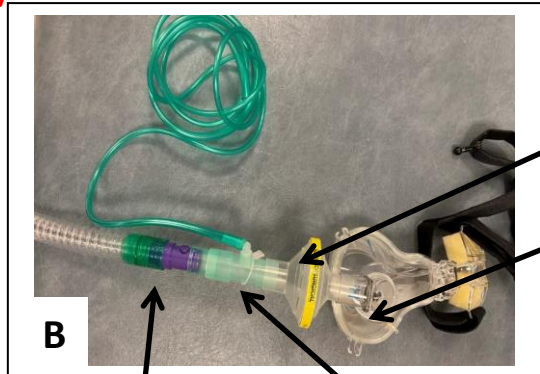

B


Filter

Mask


Expiratory Valve – DO NOT OCCLUDE O2 Entrainment

1. Connect the circuit at rear of the ventilator (see photo above photo A). Place O2 entrainment connector at end of tubing after expiratory valve and prior to bacterial filter. Attach length of green oxygen tubing from connector to the wall oxygen flowmeter (photo B).

2. Plug into mains via transformer. Device turns on.

3. Press and hold the dial on front of the device and the  button on top for at least 5-8 s.

4. **Therapy** icon appears on display. Press button centre of dial to select.

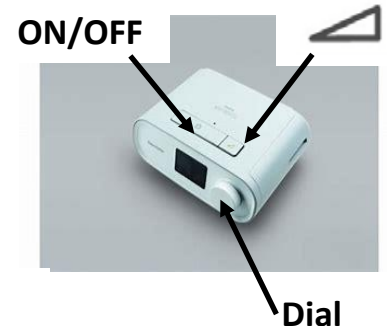

ON/OFF

Dial



5. Options include – **Mode, A-trial (off), EZ start (off), pressure.**

6. Click dial to select **Mode**. Scroll using dial to select **CPAP** and press centre of dial to confirm.

7. Using dial scroll to **pressure**. Scroll using dial to set pressure in cmH2O – **currently recommended 10 cmH2O**. Press centre of the dial to confirm selection.

8. Scroll using the dial to the **return arrow**, press the centre of the dial to confirm.

9. Set to O2 flow 15 litres and titrate down to achieve target SpO2.

10. To start and stop treatment, press the **on/off** button once.

11. To turn device off turn off at mains.



CPAP display

O2 Litres	Approx. FiO ₂
0	21%
1	24%
3	32%
5	36%
7	41%
10	54%
15	62%

Troubleshooting:

Screen dim – automatic screen saver – press dial to return to display

Need help contact respiratory team on call

The machine automatically locks and to change CPAP pressure user needs to unlock device as per step 3 to change however the device needs to be turned off first.